



REMEDIOUS®

NEURAL RETUNING REMEDIES

„Peripheral neuromodulation for the future“



CE 2274

Remedius®

NEURAL RETUNING REMEDIES

- Founded by experts in the field of pain, function and neuromodulation
- Dedicated to designing and producing innovative products for pain control and improving body function
- UK based company
- Creating a comprehensive family of devices and accessories for peripheral external and minimally invasive neuromodulation
- Created to meet the needs of physician & patient
- Products manufactured and assembled in Poland



<https://remediusneuromodulation.co.uk>



+44 (0) 20 7603 7353



info@remediusneuromodulation.co.uk



REMEDIUS Ltd
68 Alleyn Road
London SE21 8AH
United Kingdom

REMEDIUS Ltd (sp. z o.o) Poland
00-033 Warszawa
ul. W. Górskiego 6/97
POLAND
KRS: 0000902304



Teodor Goroszeniuk-REM
Wojciecha Górskiego 6/97, 00-033 Warsaw, Poland
NIP: 5252934590

Remedius® ExStim PRO



The Remedius ExStim device serves as a multi-purpose stimulator with a dual function mode for:

1. External, non-invasive stimulation with integrated Duo Stimulating Wand or attachable Remedius Stimulating Pen.
2. Minimally invasive (percutaneous) stimulation. For this purpose, minimally invasive stimulating needles or probes / catheters.

Indication of Use for ExStim PRO & Stimulating PEN

The Remedius ExStim PRO device is designed for multiple clinical applications and provides electrical stimulation of neural structures for:

- location of neural structures (nerves, plexuses and peripheral neuroreceptors)
- treatment of acute/chronic pain and functional improvement by the means of transcutaneous electrical nerve stimulation (TENS) and percutaneous electrical nerve stimulation (PENS)

Intended for adult patients and children over 5 years of age with following problems: musculoskeletal pain and dysfunction, incontinence, gastrointestinal dysfunctions, abdominal pain.

The device is intended to be used in a medical facility, by a medical practitioner.

STIMULATION PARAMETERS

- Frequency range (1-100Hz)
- Current range (40mA External; 20mA Invasive)
- Pulse width (Narrow, Medium, Wide)
- 4 polarity modes

FEATURES

- Handheld easy to use portable device
- Compact and light
- In-built Duo Stimulating wands for wireless neurostimulation
- Protective cover for wands
- Dual function for non-invasive & minimally-invasive stimulating accessories
- High quality finish
- Wide range of stimulating parameters
- In-built timer

Sites of Application for ExStim & Stimulating PEN





REMEDIOUS[®]

NEURAL RETUNING REMEDIES